

HOW TO WEAR A FACE MASK



Wash your hands before handling the mask.



Grab the mask by the elastics.



Place the elastics behind your ears.



Adjust the mask on your nose and chin.



Pinch the bridge of your nose.



You are now correctly wearing a face mask.

HOW TO REMOVE A FACE MASK



Wash your hands before handling the mask.



Grab the mask by the elastics.



Take off the mask without touching the fabric.



Put the dirty mask in a bag in order to wash it later.



Wash your hands and the soap bottle.



You have now correctly removed your face mask.

HOW TO WASH A FACE MASK

Always inspect the mask before use. Throw away a damaged mask in a closed plastic bag.



Avoid contact between a dirty mask and any clothing.



Make sure you are protected before washing the mask.




30 min



Wash with normal detergent at 60°C for minimum 30 min.



You can use a washing bag.

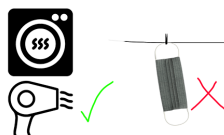

Dry using a dryer of hairdryer, no not let air dry.



You have now correctly washed your face mask.