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EN	EN	EN	EN
Down the App	1. IOS system connecting	Product parameters	Charging the unit
Please search on App Store / Google Playfor HND to download and install.	Power button/ Indicator light	Product name Ear cleaner camera Antenna Built in antenna Working frequency 2.4Ghz	1. Press and hold the button to confirm shutdown before recharging.
1 Md 6		Transmission speed 30fps Image transmitter CMOS	 The indicator light is red and constantly on while charging. The indicator light goes out when fully
Free Linesson (1990)	Press and hold the Click the icon to open Select 'WLAN Options'. power button for 2 the app and the seconds. A flashing blue interface shown in the light indicates that the figure will appear. machine is waiting for	Working temperature -10°C-50°C Battery capacity 130mAh	charged.
具新致温		Working time 35 minutes Charging time ~60Min Input current DC 5V/I50mA	Troubleshooting
		Lens inner diameter 3.6mm Focal length 15mm-20mm	1. The screen freezes A. The data transmission distance is too far, so do
Scan to get the App	Select to connect WI-FI HNDEC_55-XXXX. Go back to the app and enter above, the blue light fashes and then turns on, indicating a seccessful connection.	Note	not move away from your phone when using the product. B. Insufficient battery power can also cause freezing, so please charge it in time.
IOS After scanning the QR code, select IOS App Store to download and install, or you can search for HND in the App Store to	2. Android system connecting Power button/ Image: Click Press and hold the power button Click the icon to open the app	 If the image is unclear, wipe the lens with a cotton swab. When cleaning the lens, use a professional alcohol swab and wipe carefully. Before use, please look around. Do not use when people are walking around to prevent accidents. This product is not suitable for children under 5 	 WiFi signal cannot be transmitted/intermittent A. The lower the battery power, the weaker the WiFi signal. Fully charge the battery before use. B. The WiFi signal transmission distance has been exceeded, so the product and phone should be as close as possible. C. There are too many Wi-Fi signals in the current diag area which will affect the product's
Android After scanning the QR code, download and install the download option via a third-party browser.	for 2 seconds. A flashing blue light indicates that the materians is writing for the app to connect. Select to connect	 years old. 5. Children are not allowed to use it alone. Children must be supervised by an adult when using it. 6. Please keep the product out of reach of children. 7. Do not use the endoscope in liquid to avoid damage caused by flooding. 8. This product is only for personal care and cannot be used as a medical device. 9. Please store the product in a cool, dry place away from fire and corrosive substances. 	 signal. Try to use it in an area with fewer Wi-Fi signals. 3. No picture display A. Check whether the mobile phone is connected to the correct product Wi-Fi name. B. After confirming the connection, the product's blue light should be on constantly. If there is still no display, close the app and product, and then reopen them to connect again.
22	WI-FI HNDEC_55-XXXX. where be legath factors and then turns on, indicating a successful connection. 23	24	25

DECLARATION OF CONFORMITY
Confinity NV, Dorp 16, 9830, Sint-Martens-Latem, Belgium
declares the following device in sole responsibility:
Brand name: Wellhealth Product type: Ear cleaner camera Item number: OP_017294
ENIEC 55014-1:2021EN IEC 55014-2:2021ENIEC 61000-3-2:2019+A1:2021EN 61000-3-3:2013+A1:2019+A2:2021
Signed for and on behalf of:
Sint-Martens-Latem, Belgium April 2025
A. Pappijn - Product Manager
J. opport
Made in China