Well Health

OWNER'S MANUAL

Wellhealth in-ear hearing aid

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Intended Use

Thank you for the purchase of our hearing aid.

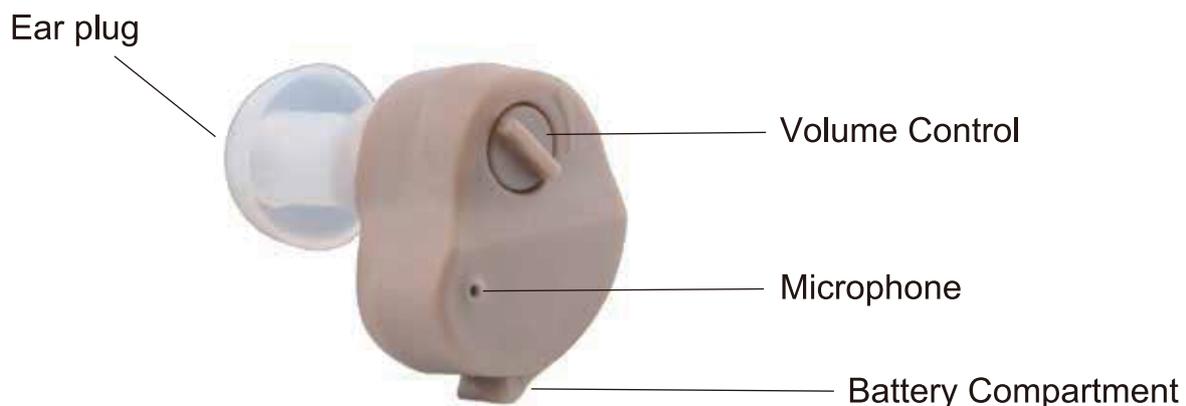
If you find it's difficult to hearing very well from others, a hearing aid can help you regain your hearing so you can live the life you are used to.

Wearing a hearing aid means rediscovering the pleasure of gathering with friends and family, feeling more confident at work, and enjoying movies, music and TV programs.

Specification

1. Max sound output: 110 ± 5 dB
2. Sound gain: ≥ 30 dB
3. Total harmonic wave distortion: $\leq 10\%$
4. Frequency range: 300Hz-4000Hz
5. Input noise: ≤ 40 dB
6. Voltage: D.C. 1.5V
7. Current: ≤ 4 mA

Specification

Battery: AG3 or A312 Zinc Air Battery

"+" Positive side of Battery

"-" Negative side of Battery

" \curvearrowright " is Volume grade.

Operation

1. Open the battery compartment, put into the corresponding battery and close the battery compartment tightly.

(Note: Open the battery compartment, it will stop working, close the battery compartment completely, It will start to work.)

2. Insert the earplug to the hearing aid, put the hearing aid in your ear and move it up and down to insure it on the right position, adjust the volume to get clear sound.
3. If you feel the whistling sound, please make sure if you put the earplug tightly, or if you use a appropriate size earplug.
4. Please take out the battery or turn off the hearing aid when you stop to use the hearing aid.

(Note: Be sure the hearing aid is in "Off" position or volume is in the minimum volume of "1", otherwise the instant sound will make you feel uncomfortable)

Maintenance

The hearing aid doesn't work. Check as follows:

1. Hearing aid is power off.
2. The battery is in correct direction.
3. Run out of battery.
4. The earphone is damaged.
5. The earphone is broken.

Note

1. Please don't use the hearing aid at a high volume for a long time. It will damage your hearing.
2. Please maintain the product all the time in order to avoid the earplug and microphone opening being blocked by foreign objects.
3. Please don't use when showering, swimming, raining or wet environment, please don't expose the hearing aid under sunshine, get close to a stove or other heat sources.
4. Please protect the hearing aid carefully and avoid dropping it on the floor.
5. Please don't remove old batteries in the hearing aid for a long time in order to avoid possible erosion caused by battery leakage.
6. Please handle the old batteries environmentally friendly.
7. Keep the hearing aid away from the children.

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